

Get complete information about the ECG Dongle Full device
and the service "CardioCloud"

Buy ECG Dongle Full and supplies

Get service support

www.cardio-cloud.ru

+7 499 608-01-88

Personal doctor:

Personal doctor ID:



Moscow, Varshavskoye highway, 125
+7 499 130-98-92

Moscow region, Dubna, Programmistov st., 4
+7 499 608-01-88




PERSONAL ELECTROCARDIOGRAPH

ECG Dongle Full

Cardio dongle
Mobile app
Cloud service

12-channel device for
professional use

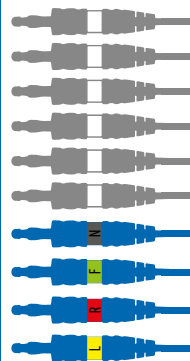

1

Download the ECG Dongle mobile app

First download and install the application "ECG Dongle" on your mobile device from official Google Play or App Store services.

App
StoreGoogle
Play

2

Attach electrodes to wires and fix it on the body

Place electrodes on the body according to the color-coded scheme.

3

Connect ECG Dongle Full to the mobile device

Connect ECG Dongle Full to the micro-USB connector on your mobile device using the OTG-cable, or directly to a USB connector, if exists.



4

Take an examination

Take a comfortable position, relax and wait for stabilization of tracing ECG on the screen. Launch the application and follow the prompts.

5

Send the data to "CardioCloud" or save it to pdf format

In the "CardioCloud" service, an expert cardiologist analyzes the ECG, supplies an opinion on how your heart working and gives necessary recommendations.

CardioCloud

www.cardio-cloud.ru